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## Eyewear maintenance

Keloa's eyewear is made of Finnish birch bark and reindeer antler. The frame has been oiled several times during manufacturing and if used regularly, it rarely requires new oiling. The frame is resistant to rain and lukewarm water. It is normal for the frame tone to deepen during use, especially in areas that are in constant contact with skin. However, this does not weaken the frame. Make-up may accumulate on the nose piece, but it will come off with proper washing. Eyewear is not to be used in sauna or soaked in water. Continuous use of hairspray is not recommended, as hairspray dries out and thus weakens all eyewear.

### Cleaning instructions:

1. Rinse glasses under running lukewarm water.
2. Spray eyeglass cleaning spray on the front and back of the lenses. Gently rub the cleaning spray on the lens with your fingertips to remove the grease that has accumulated on the lenses. For example, Piilosein Lens Spray works well. If make-up has accumulated on the nose piece, it can also be washed with eyeglass cleaning liquid and lightly massaged with a soft cotton.
3. Rinse again in running lukewarm water and dry with a soft paper.
4. Finish cleaning with the microfiber cloth that comes with the frame. Microfiber cloth can be washed in a washing machine without